



# Rust and Stain Remover

Clean Start Chemicals

Specifically designed for Skytron washer-disinfector systems.



# Rust and Stain Remover

## Cleaning and Disinfection Solution

Skytron's Rust and Stain Remover effectively eliminates rust and stains from metal surfaces, and no rinsing is necessary once the surface is dry.

### Soaking Directions:

1. Add one part of concentrate to ten (10) parts water.
2. Soak for 15 minutes or until metal surface is clean. Periodic agitation will help speed up the process.
3. Remove parts and rinse with water.

### Brush Application:

1. Add one part of concentrate to ten (10) parts water.
1. Apply liberally to the metal surface, brush, and let stand for 10 minutes.
2. Brush again and rinse with water

### Packaging:

- Four 1 gallon containers — CSTART-0701

Skytron provides safe and efficient healthcare solutions delivered through a local representative network that values long-term relationships and comprehensive service.

Focused on providing the highest quality solutions while delivering the lowest cost of ownership, we prioritize transparency and accountability in our actions.

**To learn more, visit our website at [www.skytron.com](http://www.skytron.com)**



5085 Corporate Exchange Blvd. SE  
Grand Rapids, MI 49512  
1.800.SKYTRON (759-8766)  
email: [info@skytron.com](mailto:info@skytron.com)  
[www.skytron.com](http://www.skytron.com)



REV1 12-23  
PN L2-010-245